HOBAGEL[®] PLUS

GEL COADIUVANTE

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HOBAGEL[®] PLUS

ANCILLARY GEL
for re-epithelialization processes and healing of the oral mucosa in subjects who are still developing

Junior

7 ml e

EN

DESCRIPTION

HOBAGEL PLUS JUNIOR is a biocompatible gel to be used as a support for the physiological restoration processes of oral mucous membranes and gingival tissues in children and adolescents.

INDICATIONS

HOBAGEL PLUS JUNIOR functions through its physical, mechanical and therapeutic protective action applicable to cases of:

- CHILDHOOD AND JUVENILE PERIODONTOLOGY (gingivitis in childhood and adolescence, causal and regenerative therapy of juvenile periodontitis)
- ORAL SURGERY (protection of surgically treated tissues, prevention of post-extraction complications, orthodontic surgery, frenectomy)
- INFANT AND JUVENILE STOMATOLOGY (ulcerative and / or bullous lesions of the oral mucosa, aphthosis of various types, infantile oral candidiasis (thrush), primary herpetic stomatitis and herpes labialis)
- TRAUMATOLOGY (accidental trauma to skin and oral mucous membranes, irritation during orthodontic treatments, injuries from infantile harmful habits, pyogenic granuloma)
- DENTAL PREVENTION (inhibition of the formation of oral biofilms, maintenance after healing of infantile periodontal lesions)

COMPOSITION

Calcium/Sodium Pvm/Ma Copolymer, Cellulose Gum, Paraffinum Liquidum, Petrolatum, Sodium Hyaluronate, Hydrolyzed Hyaluronic Acid, Cetylpyridinium Chloride, Ammonium Glycyrhizate, Allantoin, Panthenol, Glycyrhretinic Acid, Melaleuca Alternifolia Leaf Oil, Bisabolol, Hexetidine, Eucalyptus Globulus Leaf Oil, Commiphora Myrrha Oil, Leptospermum Scoparium Branch/Leaf Oil, Thymus Vulgaris Oil, Aroma, Tocopheryl Acetate.

HOW TO USE

- Spread a dose of the product on a clean finger and apply directly on the area to be treated, distributing evenly
- Do not massage
- The treatment must be repeated 2-3 times a day until resolved
- Avoid eating food for 30 minutes after treatment

INCOMPATIBILITY AND CONTRAINDICATIONS

- No incompatibility phenomena are known
- Contraindicated in case of ascertained hypersensitivity to one or more components

WARNINGS

- Only use the product if the packaging is intact
- Do not use after the expiration date
- Close the container immediately after use
- Keep at room temperature
- Store away from moisture
- Avoid contact with eyes
- For external use only

IMPORTANT: *HOBAGEL PLUS JUNIOR, even if indicated for children and adolescents, it must only be applied by individuals who are able to understand and implement the methods of use. In the case of infants or children, the product must be applied by parents, guardians or other adult figures who take care of these matters. The application requires small strokes of the product on the areas to be treated. In such conditions the product is harmless if ingested. When not being used, keep out of reach of children.*

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Hobama S.r.l. - Piazza della Repubblica 26 - 20124 MILANO